I. C. MEDICAL, INC.

SHIPPING INSTRUCTION LABEL FOR SENSOR PRODUCT(S)

I.C. Medical, Inc.
15002 North 25th Drive
Phoenix, Arizona 85023 USA
Phone: (623) 780-0700 (USA)
Fax: (623)780-0887 (USA)

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Made in USA

RF SENSORS

INTENDED USE: The RF Sensor is used as an accessory to the smoke evacuator, to activate it automatically, when cutting, smoke producing devices are active.

GENERAL WARNINGS:

1. Check the product for damage before use. If damaged, DO NOT use this product.
2. Protect this product from any form of mechanical damage! DO NOT throw! DO NOT use force!
3. Use only under the direction of a licensed physician.
4. The Smoke Evac and Accessory products are intended for use only with other IC Medical products. Using any other Smoke Evac or accessory not supplied by IC Medical may cause damage and/or cause the system to be inoperable and may void the warranty.

WARRANTY:

Warranty period is 2 months from date of sale.

Without limitation, the warranty does not cover damage caused by customer misuse or abuse of a product.

RF SENSOR:

For use with all Crystal Vision® Models

INSTALLATION/OPERATION:

RF Sensor when used with Monopolar Devices

(Figure 1 or 2):

1. Plug the RF Sensor's connector into Crystal Vision Smoke Evacuator's ESU/Laser port.
2. Place the RF Sensor, on the monopolar device's cable (e.g. PenEvac), by running it through the sensor's wire clip.
3. Make sure the sensor is installed on top of the wire, with the clip side facing down, close to the monopolar device's plug, to prevent capture of residual RF signal from surrounding devices.
4. Plug the monopolar device (e.g. PenEvac), into the Monopolar port of the ESU Generator.
5. Select the "Monopolar" option on ESU Generator.
6. Set ESU Generator's Cut and Coag value.
7. Activate the monopolar device by depressing cut or coag button. When the monopolar device activates, the RF Sensor should turn ON the Crystal Vision Smoke Evacuator, automatically.
8. When not in use, put away the RF Sensor by attaching the Velcro side of the RF Sensor to the smoke evacuator. Perform first-time installation by peeling the back film from the Velcro loop tape, pressing it firmly to a clean and dry area on the side of the Smoke Evacuator. To reuse the RF Sensor simply pull it off from the Smoke Evacuator's side.

RF Sensor when used with Bipolar Devices

(Figure 3 or 4):

1. Plug the RF Sensor's connector into Crystal Vision Smoke Evacuator's ESU/Laser port.
2. Place the RF Sensor, on the bipolar device's cable, by running it through the sensor's wire clip.
3. Make sure the sensor is installed on top of the wire, with the clip side facing down, close to the bipolar device's plug, to prevent capture of residual RF signal from surrounding devices.
4. Plug the bipolar device into the Bipolar Port of the ESU Generator.
5. Select the "Bipolar" option on ESU Generator.
6. Activate the bipolar device. When the bipolar device activates, the RF Sensor should turn ON the Crystal Vision Smoke Evacuator, automatically.
7. When not in use, put away the RF Sensor by attaching the Velcro side of the RF Sensor to the smoke evacuator. Perform first-time installation by peeling the back film from the Velcro loop tape, pressing it firmly to a clean and dry area on the side of the Smoke Evacuator. To reuse the RF Sensor simply pull it off from the Smoke Evacuator's side.

UNIVERSAL BLUE RF SENSOR

For use with all Crystal Vision® Models

INSTALLATION/OPERATION:

UNIVERSAL BLUE™ RF Sensor when used with Monopolar Devices

(Figure 5 or 6):

1. Plug the **UNIVERSAL BLUE™** RF Sensor's connector into Crystal Vision Smoke Evacuator's ESU/Laser port.
2. Place the **UNIVERSAL BLUE™** RF Sensor, on the monopolar device's cable (e.g. PenEvac), making sure the sensor is installed on top of the cable, facing down, to prevent capture of residual RF signal from surrounding devices.
3. Use both of the self-adhering Velcro straps to wrap around the sensor and over the monopolar device's cable, to secure the cable to the sensor at both ends of the sensor.
4. Plug the monopolar device (e.g. PenEvac), into the Monopolar port of the ESU Generator.
5. Select the "Monopolar" option on ESU Generator.
6. Set ESU Generator's Cut and Coag value.
7. Activate the monopolar device by depressing cut or coag button. When the monopolar device activates, the RF Sensor should turn ON the Crystal Vision Smoke Evacuator, automatically.

UNIVERSAL BLUE™ RF Sensor when used with Bipolar Devices

(Figure 7 or 8):

1. Plug the **UNIVERSAL BLUE™** RF Sensor's connector into Crystal Vision Smoke Evacuator's ESU/Laser port.
2. Place the **UNIVERSAL BLUE™** RF Sensor, on the bipolar device's cable, making sure the sensor is installed on top of the cable, facing down, to prevent capture of residual RF signal from surrounding devices.
3. Use both of the self-adhering Velcro straps to wrap around the sensor and over the bipolar device's cable, to secure the cable to the sensor at both ends of the sensor.
4. Plug the bipolar device into the ESU Generator.
5. Select the "Bipolar" option on ESU Generator.
6. Activate the bipolar device. When the bipolar device activates, the RF Sensor should turn ON the Crystal Vision Smoke Evacuator, automatically.

UNIVERSAL BLUE™ RF Sensor when used with Harmonic Scalpel Generator

(Figure 9 or 10):

1. Plug the **UNIVERSAL BLUE™** RF Sensor's connector into Crystal Vision Smoke Evacuator's ESU/Laser port.
2. Place the **UNIVERSAL BLUE™** RF Sensor on the hand piece cable, making sure the sensor is installed on top of the cable, facing down, to prevent capture of residual RF signal from surrounding devices.
3. Use both of the self-adhering Velcro straps to wrap around the sensor and over the hand piece cable, to secure the cable to the sensor at both ends of the sensor.
4. Plug the handpiece into the Harmonic Scalpel Generator.
5. Set the Harmonic Scalpel Generator
6. Activate the handpiece. When the handpiece is activated, the **UNIVERSAL BLUE™** RF Sensor should turn ON the Crystal Vision Smoke Evacuator automatically.

SERIOUS ADVERSE EVENTS:

Any serious adverse event or incident that occurs in relation to the device or accessory should be reported to the manufacturer, I.C. Medical, Inc., at complaints@icmedical.com and to the FDA. In addition, European customers should also report to the Authorized Representative at the address listed on the label or IFU and to the competent authority in the member state.

This product is covered by the following U.S. patent: 5,318,516. Other U.S. and/or foreign patents may be pending.

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