
I. C. MEDICAL, INC.

FOOT SWITCH INSTALLATION INSTRUCTIONS

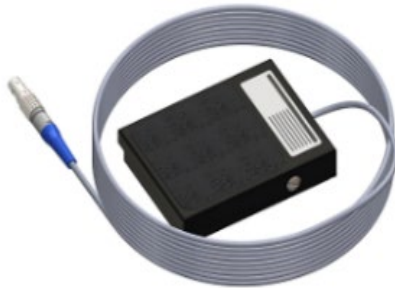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MDSS-UK RP Ltd.
6 Wilmslow Road,
Rusholme,
Manchester M14 5TP,
United Kingdom

Made in the USA

Label: ICM-001-1271 Rev. F Date: 2025-05

INDICATIONS FOR USE: The Foot Switch is used as an accessory to the smoke evacuator to turn the smoke evacuator ON and OFF.

TECHNICAL SPECIFICATION:

I.C. Medical, Inc. recommends the use of Foot Switch with the Crystal Vision® Smoke Evacuators-all models. The Foot Switch is used for end-user convenience to activate the smoke evacuator when needed. The foot switch connects to Crystal Vision® Smoke Evacuators through the same input connector used by the Laser and RF Sensor(s).

TRANSPORT AND STORAGE CONDITIONS:

Temperature limits: -40°C to 70°C (-40°F to 158°F)
Relative Humidity limitation: 10% to 100%.

INSTALLATION/OPERATION:

For use with all Crystal Vision® models

If you are going to use your CRYSTAL VISION® with a foot switch proceed with the following:

1. Place the foot switch into the connector marked "LASER" on the Smoke Evacuator front-panel (You may keep the ESU Sensor plugged in if desired). Both LASER & ESU input connectors are identical, and both will accept all standard Smoke Evacuator sensor assemblies.
2. Press the foot switch to activate the Smoke Evacuator. The Smoke Evacuator will now turn on any time the foot switch is activated. Flow will continue until pressure on the footswitch is removed and the allowed delay time is complete.

GENERAL WARNINGS:

1. Check the product for damage before use. If damaged, DO NOT use this product.
2. Protect this product from any form of mechanical damage! DO NOT throw! DO NOT use force!
3. Use only under the direction of a licensed physician.
4. The Smoke Evac and Accessory products are intended for use only with other IC Medical products. Using any other Smoke Evac or accessory not supplied by IC Medical may cause damage and/or cause the system to be inoperable and may void the warranty.

SERIOUS ADVERSE EVENTS:

Any serious adverse event or incident that occurs in relation to the device or accessory should be reported to the manufacturer, I.C. Medical, Inc., at complaints@icmedical.com and to the FDA. In addition, European customers should also report to the Authorized Representative at the address listed on the label or IFU and to the competent authority in the member state.

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