Large Coconut Charcoal Output Filter Instructions for Use

I. C. Medical, Inc.
 15002 North 25th Drive
 Phoenix, Arizona 85023 USA
 Phone: (623) 780-0700 (USA)
 Fax: (623)780-0887 (USA)
 www.icmedical.com

EU REP **CE** **UK REP** **UK RA**
 MDSS GmbH
 Schiffgraben 41
 30175 Hannover,
 Germany


MDSS-UK RP Ltd.
 6 Wilmslow Road,
 Rusholme,
 Manchester M14 5TP,
 United Kingdom

Made in the USA

en

Large Coconut Charcoal Output Filter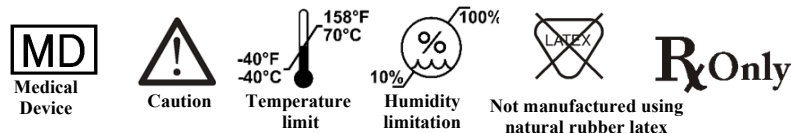

INTENDED USE: The Large Coconut Charcoal Output Filter is used as an accessory to the smoke evacuator, to reduce noise, neutralize gases, odors and remove volatile organic compounds generated by the surgical smoke.

- For use with all Crystal Vision® Models.
- Read and follow instructions in smoke evacuator manual.
- Screw filter into connector on back of the smoke evacuator.
- **Replace filter every 3 months, or whenever odors are detected.**
- **Turn OFF the unit when replacing the filter.**
- **Screw filter all the way, to prevent leakage and noise.**
- Dispose of filter in accordance with your institution's established procedures.

GENERAL WARNINGS:

- Check the product for damage before use. If damaged, DO NOT use this product.
- Protect this product from any form of mechanical damage! DO NOT throw! DO NOT use force!
- Use only under the direction of a licensed physician.
- The Smoke Evac and Accessory products are intended for use only with other IC Medical products. Using any other Smoke Evac or accessory not supplied by IC Medical may cause damage and/or cause the system to be inoperable and may void the warranty.

SERIOUS ADVERSE EVENTS:

Any serious adverse event or incident that occurs in relation to the device or accessory should be reported to the manufacturer, I.C. Medical, Inc., at complaints@icmedical.com and to the FDA. In addition, European customers should also report to the Authorized Representative at the address listed on the label or IFU and to the competent authority in the member state.

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