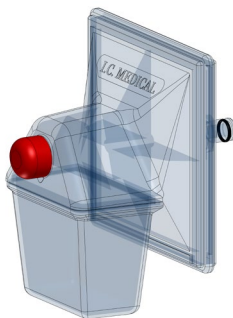
I. C. MEDICAL, INC.

SAFEGUARD BLUE® HYDROPHOBIC ULPA FILTER WITH BUILT-IN FLUID TRAP, WARNINGS, CAUTIONS & INSTRUCTIONS

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Australian Sponsor:
I.C. Medical Australia Pty Ltd
Level 1, Suite 3, 130 Bundall Road
Bundall, QLD 4217,
Australia

Made in the USA

SAFEGUARD BLUE® Hydrophobic ULPA Filter with Built-in Fluid Trap

Indications for Use:

The filter is used as an attachment for all I.C. Medical Smoke Evacuator models, to remove, filtrate the smoke plume and to collect in its built-in fluid trap, any incidental fluid, vapors, oil mist, hydrocarbon vapors, fat vapors, moisture droplets, odors, organic solvent vapors, and toxic vapors from the surgical site.

Technical Specification:

I.C. Medical, Inc. recommends the use of the SAFEGUARD BLUE® Hydrophobic ULPA Filter with Built-in Fluid Trap with the Crystal Vision® Smoke Evacuators-all models, and other I.C. Medical products.

Filtration Efficiency:

Crystal Vision® Smoke Evacuator Mode of Operation		Particles at: (in microns)		
		0.03	0.12	0.3
LAP	Efficiency (%)	>99.9999	>99.9999	>99.9999
OPEN	Efficiency (%)	>99.9999	>99.9999	>99.9999

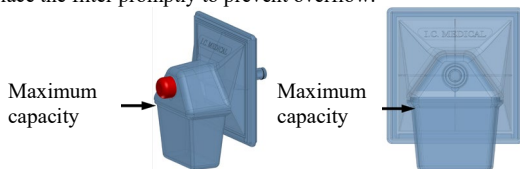
Transport and Storage Conditions:

Temperature limits: -40°C to 70°C (-40°F to 158°F);

Relative Humidity limitation: 10% to 100%.

WARNINGS:

1. Check the product for damage before use. If damaged, DO NOT use this product.
2. Protect this product from any form of mechanical damage! DO NOT throw! DO NOT use force!
3. The SAFEGUARD BLUE® Hydrophobic ULPA Filter with Built-in Fluid Trap includes a hydrophobic barrier to protect the smoke evacuator from fluid ingress. Do not exceed the fluid trap capacity. Monitor fluid levels regularly and replace the filter promptly to prevent overflow.

4. Use only under the direction of a licensed physician.

CAUTIONS:

1. Read Operating and Installation Manual for Smoke Evacuator.
2. Before installing or removing any filter, be sure that the system is turned off and the electrical plug has been disconnected.
3. Failure to install the SAFEGUARD BLUE® Hydrophobic ULPA Filter with Built-in Fluid Trap could result in contamination and possible electrical damage of the I. C. Medical Smoke Evacuator.
4. **RX ONLY** Federal Law restricts this device to sale by or on the order of a licensed physician.

INSTRUCTIONS:

1. The SAFEGUARD BLUE® Hydrophobic ULPA Filter with Built-in Fluid Trap should only be used with I.C. Medical Smoke Evacuators (all models).
2. Attach the filter assembly to the smoke evacuator and attach the accessory tubing to the filter assembly.
3. Replace the filter assembly when “Change Filter” indicator comes on.
4. Replace the filter assembly when desired levels of gas flow cannot be achieved.
5. Replace the filter assembly if clogged or before fluid reaches the trap’s capacity.
6. Replace the filter assembly after any procedures when moisture or fluids are present in the water/vapor trap.
7. Replace cap on end of filter when not in use.
8. Dispose of filter according to your institution’s policy for biologically contaminated waste disposal.

9. This is a NON-STERILE product. It may be used until there is an indication to replace the filter (listed above). After that time do not attempt to clean, sterilize, or re-use this filter.

PACKAGING:

Use this product only if the packaging has not been opened or damaged. In this case throw the product away!

SERIOUS ADVERSE EVENTS:

Any serious adverse event or incident that occurs in relation to the device or accessory should be reported to the manufacturer, I.C. Medical, Inc., at complaints@icmedical.com and to the FDA. In addition, European customers should also report to the Authorized Representative at the address listed on the label or IFU and to the competent authority in the member state.

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