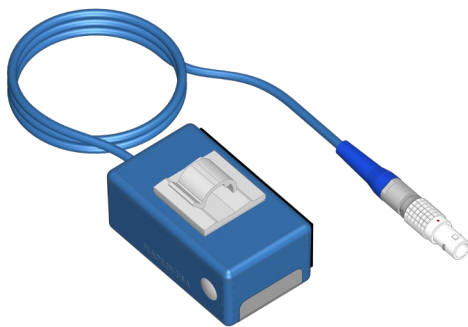
I. C. MEDICAL, INC.

SHIPPING INSTRUCTION LABEL FOR SENSOR PRODUCT(S)

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Australian Sponsor:
I.C. Medical Australia Pty Ltd
Level 1, Suite 3, 130 Bundall Road
Bundall, QLD 4217,
Australia

Made in the USA

RF SENSORS

INTENDED USE: The RF Sensor is used as an accessory to the smoke evacuator, to activate it automatically, when cutting, smoke producing devices are active.

Technical Specification:

I.C. Medical, Inc. recommends the use of RF Sensors with the Crystal Vision® Smoke Evacuators- all models.

When connected to the Crystal Vision® Smoke Evacuator and active cutting devices, the RF Sensor(s) can detect the active cutting device in the operating room and automatically activate the smoke evacuator.

Transport and Storage Conditions:

Temperature limits: -40°C to 70°C (-40°F to 158°F);

Relative Humidity limitation: 10% to 100%.

GENERAL WARNINGS:

1. Check the product for damage before use. If damaged, DO NOT use this product.
2. Protect this product from any form of mechanical damage! DO NOT throw! DO NOT use force!
3. Use only under the direction of a licensed physician.
4. The Smoke Evac and Accessory products are intended for use only with other IC Medical products. Using any other Smoke Evac or accessory not supplied by IC Medical may cause damage and/or cause the system to be inoperable and may void the warranty.

WARRANTY:

Warranty period is 2 months from date of sale.

Without limitation, the warranty does not cover damage caused by customer misuse or abuse of a product.

RF SENSOR:

For use with all Crystal Vision® Models

INSTALLATION/OPERATION:

RF Sensor when used with Monopolar Devices

(Figure 1 or 2):

1. Plug the RF Sensor's connector into Crystal Vision Smoke Evacuator's ESU/Laser port.
2. Place the RF Sensor, on the monopolar device's cable (e.g. PenEvac), by running it through the sensor's wire clip.
3. Make sure the sensor is installed on top of the wire, with the clip side facing down, close to the monopolar device's plug, to prevent capture of residual RF signal from surrounding devices.
4. Plug the monopolar device (e.g. PenEvac), into the Monopolar port of the ESU Generator.
5. Select the "Monopolar" option on ESU Generator.
6. Set ESU Generator's Cut and Coag value.
7. Activate the monopolar device by depressing cut or coag button. When the monopolar device activates, the RF Sensor should turn ON the Crystal Vision Smoke Evacuator, automatically.
8. When not in use, put away the RF Sensor by attaching the Velcro side of the RF Sensor to the smoke evacuator. Perform first-time installation by peeling the back film from the Velcro loop tape, pressing it firmly to a clean and dry area on the side of the Smoke Evacuator. To reuse the RF Sensor simply pull it off from the Smoke Evacuator's side.

RF Sensor when used with Bipolar Devices

(Figure 3 or 4):

1. Plug the RF Sensor's connector into Crystal Vision Smoke Evacuator's ESU/Laser port.
2. Place the RF Sensor, on the bipolar device's cable, by running it through the sensor's wire clip.
3. Make sure the sensor is installed on top of the wire, with the clip side facing down, close to the bipolar device's plug, to prevent capture of residual RF signal from surrounding devices.
4. Plug the bipolar device into the Bipolar Port of the ESU Generator.
5. Select the "Bipolar" option on ESU Generator.
6. Activate the bipolar device. When the bipolar device activates, the RF Sensor should turn ON

the Crystal Vision Smoke Evacuator, automatically.

7. When not in use, put away the RF Sensor by attaching the Velcro side of the RF Sensor to the smoke evacuator. Perform first-time installation by peeling the back film from the Velcro loop tape, pressing it firmly to a clean and dry area on the side of the Smoke Evacuator. To reuse the RF Sensor simply pull it off from the Smoke Evacuator's side.

UNIVERSAL BLUE RF SENSOR

For use with all Crystal Vision® Models

INSTALLATION/OPERATION:

UNIVERSAL BLUE™ RF Sensor when used with Monopolar Devices

(Figure 5 or 6):

1. Plug the **UNIVERSAL BLUE™** RF Sensor's connector into Crystal Vision Smoke Evacuator's ESU/Laser port.
2. Place the **UNIVERSAL BLUE™** RF Sensor, on the monopolar device's cable (e.g. PenEvac), making sure the sensor is installed on top of the cable, facing down, to prevent capture of residual RF signal from surrounding devices.
3. Use both of the self-adhering Velcro straps to wrap around the sensor and over the monopolar device's cable, to secure the cable to the sensor at both ends of the sensor.
4. Plug the monopolar device (e.g. PenEvac), into the Monopolar port of the ESU Generator.
5. Select the "Monopolar" option on ESU Generator.
6. Set ESU Generator's Cut and Coag value.
7. Activate the monopolar device by depressing cut or coag button. When the monopolar device activates, the RF Sensor should turn ON the Crystal Vision Smoke Evacuator, automatically.

UNIVERSAL BLUE™ RF Sensor when used with Bipolar Devices

(Figure 7 or 8):

1. Plug the **UNIVERSAL BLUE™** RF Sensor's connector into Crystal Vision Smoke Evacuator's ESU/Laser port.
2. Place the **UNIVERSAL BLUE™** RF Sensor, on the bipolar device's cable, making sure the sensor is installed on top of the cable, facing down, to prevent capture of residual RF signal from surrounding devices.
3. Use both of the self-adhering Velcro straps to wrap around the sensor and over the bipolar device's cable, to secure the cable to the sensor at both ends of the sensor.
4. Plug the bipolar device into the ESU Generator.
5. Select the "Bipolar" option on ESU Generator.
6. Activate the bipolar device. When the bipolar device activates, the RF Sensor should turn ON the Crystal Vision Smoke Evacuator, automatically.

UNIVERSAL BLUE™ RF Sensor when used with Harmonic Scalpel Generator

(Figure 9 or 10):

1. Plug the **UNIVERSAL BLUE™** RF Sensor's connector into Crystal Vision Smoke Evacuator's ESU/Laser port.
2. Place the **UNIVERSAL BLUE™** RF Sensor on the hand piece cable, making sure the sensor is installed on top of the cable, facing down, to prevent capture of residual RF signal from surrounding devices.
3. Use both of the self-adhering Velcro straps to wrap around the sensor and over the hand piece cable, to secure the cable to the sensor at both ends of the sensor.
4. Plug the handpiece into the Harmonic Scalpel Generator.
5. Set the Harmonic Scalpel Generator
6. Activate the handpiece. When the handpiece is activated, the **UNIVERSAL BLUE™** RF Sensor should turn ON the Crystal Vision Smoke Evacuator automatically.

SERIOUS ADVERSE EVENTS:

Any serious adverse event or incident that occurs in relation to the device or accessory should be reported to the manufacturer, I.C. Medical, Inc., at complaints@icmedical.com and to the FDA. In addition, European customers should also report to the Authorized Representative at the address listed on the label or IFU and to the competent authority in the member state.

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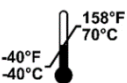
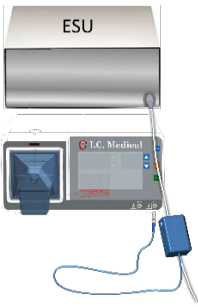
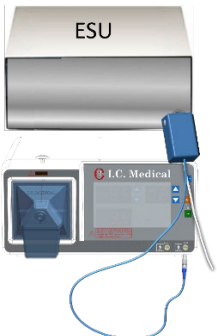
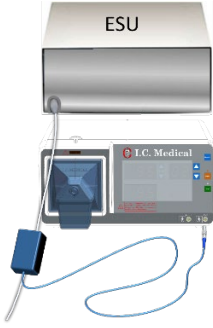
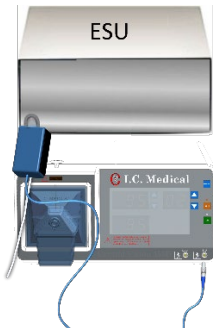
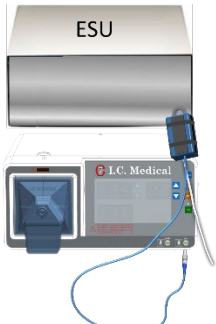
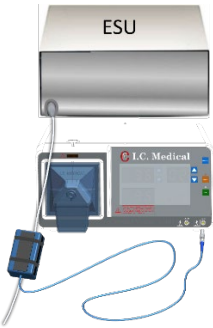
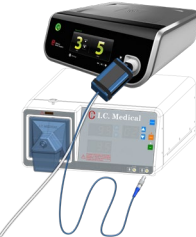
Figures RF Sensor	Languages	Figures	“ESU” translation
 The image shows the ESU RF Sensor device, which consists of a white rectangular unit with a blue cable attached to its side. The cable is connected to a blue rectangular sensor unit. The device is labeled 'ESU' and 'I.C. Medical'.	English (en)	Figure 1	Electrosurgical Generator
	French (fr)	Figure 1	Générateur électrochirurgical
	German (de)	Abb. 1	Elektrochirurgische her Generator
	Romanian (ro)	Figura 1	Generator de Electrochirurgie
	Russian (ru)	Рисунок 1	Электрохирургический генератор
	Spanish (es)	Figura 1	Generador electroquirúrgico
	Italian (it)	Figura 1	Generatore Elettrochirurgico
	Turkish (tr)	Şekil 1	Elektrocerrahi jeneratörü
	Portuguese (pt)	Figura 1	Gerador eletrocirúrgico
	Slovenian (sl)	Slika 1	Elektrokirurški generator
	Croatian (hr)	Slika 1	Elektrokirurški generator
	Czech (cz)	Postava 1	Elektrochirurgický generator
	Slovak (sk)	Obrázok 1	Elektrochirurgický generator
	Greek (gr)	Εικόνα 1	Ηλεκτροχειρουργική Γεννήτρια
 The image shows the ESU RF Sensor device, which consists of a white rectangular unit with a blue cable attached to its side. The cable is connected to a blue rectangular sensor unit. The device is labeled 'ESU' and 'I.C. Medical'.	English (en)	Figure 2	Electrosurgical Generator
	French (fr)	Figure 2	Générateur électrochirurgical
	German (de)	Abb. 2	Elektrochirurgische her Generator
	Romanian (ro)	Figura 2	Generator de Electrochirurgie
	Russian (ru)	Рисунок 2	Электрохирургический генератор
	Spanish (es)	Figura 2	Generador electroquirúrgico
	Italian (it)	Figura 2	Generatore Elettrochirurgico
	Turkish (tr)	Şekil 2	Elektrocerrahi jeneratörü
	Portuguese (pt)	Figura 2	Gerador eletrocirúrgico
	Slovenian (sl)	Slika 2	Elektrokirurški generator
	Croatian (hr)	Slika 2	Elektrokirurški generator
	Czech (cz)	Postava 2	Elektrochirurgický generator
	Slovak (sk)	Obrázok 2	Elektrochirurgický generator
	Greek (gr)	Εικόνα 2	Ηλεκτροχειρουργική Γεννήτρια

Figures RF Sensor	Languages	Figures	“ESU” translation
 The image shows an ESU (Electrosurgical Unit) generator and a handpiece. The generator is a white, rectangular device with a control panel on the front. The handpiece is a blue, cylindrical device connected to the generator by a blue cable. The text 'ESU' is printed on the top of the generator.	English (en)	Figure 3	Electrosurgical Generator
	French (fr)	Figure 3	Générateur électrochirurgical
	German (de)	Abb. 3	Elektrochirurgische her Generator
	Romanian (ro)	Figura 3	Generator de Electrochirurgie
	Russian (ru)	Рисунок 3	Электрохирургический генератор
	Spanish (es)	Figura 3	Generador electroquirúrgico
	Italian (it)	Figura 3	Generatore Elettrochirurgico
	Turkish (tr)	Şekil 3	Elektrocerrahi jeneratörü
	Portuguese (pt)	Figura 3	Gerador eletrocirúrgico
	Slovenian (sl)	Slika 3	Elektrokirurški generator
	Croatian (hr)	Slika 3	Elektrokirurški generator
	Czech (cz)	Postava 3	Elektrochirurgický generator
	Slovak (sk)	Obrázok 3	Elektrochirurgický generator
	Greek (gr)	Εικόνα 3	Ηλεκτροχειρουργική Γεννήτρια
 The image shows an ESU (Electrosurgical Unit) generator and a handpiece. The generator is a white, rectangular device with a control panel on the front. The handpiece is a blue, cylindrical device connected to the generator by a blue cable. The text 'ESU' is printed on the top of the generator.	English (en)	Figure 4	Electrosurgical Generator
	French (fr)	Figure 4	Générateur électrochirurgical
	German (de)	Abb. 4	Elektrochirurgische her Generator
	Romanian (ro)	Figura 4	Generator de Electrochirurgie
	Russian (ru)	Рисунок 4	Электрохирургический генератор
	Spanish (es)	Figura 4	Generador electroquirúrgico
	Italian (it)	Figura 4	Generatore Elettrochirurgico
	Turkish (tr)	Şekil 4	Elektrocerrahi jeneratörü
	Portuguese (pt)	Figura 4	Gerador eletrocirúrgico
	Slovenian (sl)	Slika 4	Elektrokirurški generator
	Croatian (hr)	Slika 4	Elektrokirurški generator
	Czech (cz)	Postava 4	Elektrochirurgický generator
	Slovak (sk)	Obrázok 4	Elektrochirurgický generator
	Greek (gr)	Εικόνα 4	Ηλεκτροχειρουργική Γεννήτρια

Figures/ UNIVERSAL BLUE™ RF Sensor	Languages	Figures	“ESU” translation
	English (en)	Figure 5	Electrosurgical Generator
	French (fr)	Figure 5	Générateur électrochirurgical
	German (de)	Abb. 5	Elektrochirurgische her Generator
	Romanian (ro)	Figura 5	Generator de Electrochirurgie
	Russian (ru)	Рисунок 5	Электрохирургический генератор
	Spanish (es)	Figura 5	Generador electroquirúrgico
	Italian (it)	Figura 5	Generatore Elettrochirurgico
	Turkish (tr)	Şekil 5	Elektrocerrahi jeneratörü
	Portuguese (pt)	Figura 5	Gerador eletrocirúrgico
	Slovenian (sl)	Slika 5	Elektrokirurški generator
	Croatian (hr)	Slika 5	Elektrokirurški generator
	Czech (cz)	Postava 5	Elektrochirurgický generator
	Slovak (sk)	Obrázok 5	Elektrochirurgický generator
	Greek (gr)	Εικόνα 5	Ηλεκτροχειρουργική Γεννήτρια
	English (en)	Figure 6	Electrosurgical Generator
	French (fr)	Figure 6	Générateur électrochirurgical
	German (de)	Abb. 6	Elektrochirurgische her Generator
	Romanian (ro)	Figura 6	Generator de Electrochirurgie
	Russian (ru)	Рисунок 6	Электрохирургический генератор
	Spanish (es)	Figura 6	Generador electroquirúrgico
	Italian (it)	Figura 6	Generatore Elettrochirurgico
	Turkish (tr)	Şekil 6	Elektrocerrahi jeneratörü
	Portuguese (pt)	Figura 6	Gerador eletrocirúrgico
	Slovenian (sl)	Slika 6	Elektrokirurški generator
	Croatian (hr)	Slika 6	Elektrokirurški generator
	Czech (cz)	Postava 6	Elektrochirurgický generator
	Slovak (sk)	Obrázok 6	Elektrochirurgický generator
	Greek (gr)	Εικόνα 6	Ηλεκτροχειρουργική Γεννήτρια

Figures/ UNIVERSAL BLUE™ RF Sensor	Languages	Figures	“ESU” translation
 The image shows the ESU Universal Blue RF Sensor device. It consists of a white rectangular base unit labeled 'ESU' at the top. A blue cable connects the base unit to a handheld control unit. The handheld unit has a small screen and several buttons. The brand name 'I.C. Medical' is visible on the handheld unit. A blue coagulation probe is attached to the handheld unit via a blue cable.	English (en)	Figure 7	Electrosurgical Generator
	French (fr)	Figure 7	Générateur électrochirurgical
	German (de)	Abb. 7	Elektrochirurgisc her Generator
	Romanian (ro)	Figura 7	Generator de Electrochirurgie
	Russian (ru)	Рисунок 7	Электрохирургический генератор
	Spanish (es)	Figura 7	Generador electroquirúrgico
	Italian (it)	Figura 7	Generatore Elettrochirurgico
	Turkish (tr)	Şekil 7	Elektrocerrahi jeneratörü
	Portuguese (pt)	Figura 7	Gerador eletrocirúrgico
	Slovenian (sl)	Slika 7	Elektrokirurški generator
	Croatian (hr)	Slika 7	Elektrokirurški generator
	Czech (cz)	Postava 7	Elektrochirurgický generator
	Slovak (sk)	Obrázok 7	Elektrochirurgický generator
	Greek (gr)	Εικόνα 7	Ηλεκτροχειρουργική Γεννήτρια
 The image shows the ESU Universal Blue RF Sensor device, identical to the one in the first section. It consists of a white rectangular base unit labeled 'ESU' at the top. A blue cable connects the base unit to a handheld control unit. The handheld unit has a small screen and several buttons. The brand name 'I.C. Medical' is visible on the handheld unit. A blue coagulation probe is attached to the handheld unit via a blue cable.	English (en)	Figure 8	Electrosurgical Generator
	French (fr)	Figure 8	Générateur électrochirurgical
	German (de)	Abb. 8	Elektrochirurgisc her Generator
	Romanian (ro)	Figura 8	Generator de Electrochirurgie
	Russian (ru)	Рисунок 8	Электрохирургический генератор
	Spanish (es)	Figura 8	Generador electroquirúrgico
	Italian (it)	Figura 8	Generatore Elettrochirurgico
	Turkish (tr)	Şekil 8	Elektrocerrahi jeneratörü
	Portuguese (pt)	Figura 8	Gerador eletrocirúrgico
	Slovenian (sl)	Slika 8	Elektrokirurški generator
	Croatian (hr)	Slika 8	Elektrokirurški generator
	Czech (cz)	Postava 8	Elektrochirurgický generator
	Slovak (sk)	Obrázok 8	Elektrochirurgický generator
	Greek (gr)	Εικόνα 8	Ηλεκτροχειρουργική Γεννήτρια

Figures/ UNIVERSAL BLUE™ RF Sensor	Languages	Figures	“ESU” translation
	English (en)	Figure 9	Electrosurgical Generator
	French (fr)	Figure 9	Générateur électrochirurgical
	German (de)	Abb. 9	Elektrochirurgische her Generator
	Romanian (ro)	Figura 9	Generator de Electrochirurgie
	Russian (ru)	Рисунок 9	Электрохирургический генератор
	Spanish (es)	Figura 9	Generador electroquirúrgico
	Italian (it)	Figura 9	Generatore Elettrochirurgico
	Turkish (tr)	Şekil 9	Elektrocerrahi jeneratörü
	Portuguese (pt)	Figura 9	Gerador eletrocirúrgico
	Slovenian (sl)	Slika 9	Elektrokirurški generator
	Croatian (hr)	Slika 9	Elektrokirurški generator
	Czech (cz)	Postava 9	Elektrochirurgický generator
	Slovak (sk)	Obrázok 9	Elektrochirurgický generator
	Greek (gr)	Εικόνα 9	Ηλεκτροχειρουργική Γεννήτρια
	English (en)	Figure 10	Electrosurgical Generator
	French (fr)	Figure 10	Générateur électrochirurgical
	German (de)	Abb. 10	Elektrochirurgische her Generator
	Romanian (ro)	Figura 10	Generator de Electrochirurgie
	Russian (ru)	Рисунок 10	Электрохирургический генератор
	Spanish (es)	Figura 10	Generador electroquirúrgico
	Italian (it)	Figura 10	Generatore Elettrochirurgico
	Turkish (tr)	Şekil 10	Elektrocerrahi jeneratörü
	Portuguese (pt)	Figura 10	Gerador eletrocirúrgico
	Slovenian (sl)	Slika 10	Elektrokirurški generator
	Croatian (hr)	Slika 10	Elektrokirurški generator
	Czech (cz)	Postava 10	Elektrochirurgický generator
	Slovak (sk)	Obrázok 10	Elektrochirurgický generator
	Greek (gr)	Εικόνα 10	Ηλεκτροχειρουργική Γεννήτρια